

Bilanx bathing collection
strike the balance Design Safety

Making life easier
with style and confidence.

BILANX INSTALLATION INSTRUCTIONS

(for hollow wall and wood stud walls)

1. Remove Contents from box
2. Place Bilanx Grab Bar on the wall in your desired location
3. Mark the 2 elongated holes on each flange base
4. Remove Bilanx Grab Bar and drill 1/2" holes
5. Insert the 4 togglers (Sold Separately) into the four holes, Slide the plastic collar against the wall, and Snap of the excess plastic tabs
-OR-
For applications where in-wall wood backing or wall studs are present please use the stainless steel wood screws provided in place of the togglers
6. Remove the 2 flange bases from the Grab Bar by loosening the 3 set screws in each flange post
7. Place the flange bases on the wall aligning the elongated holes in the flange base with the inserted togglers
8. Attach flange bases using the 3 machine screws provided, tighten and secure
9. Slide the concealed flange covers over the Bilanx Grab Bar posts
10. Then slide the Bilanx Grab Bar posts over the flange base previously attached to the wall
11. Tighten all set screws

ENJOY YOUR NEW BILANX GRAB BAR



Health @ Home Inc.

www.healthathomeinc.com

704.960.0148 | 980.900.2845 | 888.802.8186 fax orders@healthathomeinc.com