

Flip Down Grab Bar Installation Instructions

BAR-FB29 x OD x FINISH



SAFETY NOTE: Concealed In-Wall support backing is **ALWAYS REQUIRED** for this product (not included) – and must be supplied and selected by a licensed installer/contractor to meet both weight bearing and local code requirements. **PRODUCT INSTALLATION SHOULD BE DONE BY A LICENSED INSTALLER/CONTRACTOR.**

Maintenance: Clean with mild soap and water.

Warranty – Full 1 year warranty for material defects and manufacturing defects.

CAUTION: Personal injury and/or property damage may result from improper wall support or mounting, which is not covered under warranty.

WARNING: Do **NOT** use this instruction sheet for straight grab bars, grab bars with welded stud or internal threads

IT IS ESSENTIAL TO MARK THE SCREW HOLES ON THE MOUNTING PLATE AS WELL AS THE DISTANCE FROM THE MOUNTING PLATE TO THE FINISH FLOOR TO ASSURE PROPER ALIGNMENT.

INSTALLATION STEPS:

1. Confirm contents of box: Grab Bar and screws
2. Tools & supplies needed: Phillips screw driver, pencil, drill, silicone caulk.
3. Carefully consider the following before installation:
MOUNTING LOCATION: Install only where proper structure is provided behind the wall. Measure area on the wall for the grab bar in both the up and down positions to avoid hitting other objects. Also avoid pipes, wires, grout, etc.
4. Use the mounting plate as a template, mark center of screw hole locations.
5. Drill pilot holes.
6. **IMPORTANT: MUST HAVE PROPER CONCEALED IN-WALL BACKING INSTALLED BY A LICENSED INSTALLATION PROFESSIONAL.**
7. On the backside of the mounting plate, where it will touch the wall, apply a bead of silicone caulk around each screw hole.
8. Install the bar with the manufacturer's provided hardware. After installing the bar, inspect each screw to ensure it is tight, straight and flush with the flange. **Warning: Avoid damage to wall, tile, or fasteners by not over-tightening screws.**
9. Test grab bar up and down movement.

ORDER ENTRY BY FAX 888.802.8186 OR EMAIL orders@healthathomeinc.com

Office 704.960.0148 or 980.900.2845 www.healthathomeinc.com

Products that enhance safety, vitality, & longevity at home.